

RISP




Getting started

It is easy to get started with RISP. There is no need to install any additional software



Your local implementation RISP team may have already granted access to all clinicians in the Trust.

You may see a 'RISP' shortcut on your desktop or the link for RISP on your intranet from where you access other clinical systems

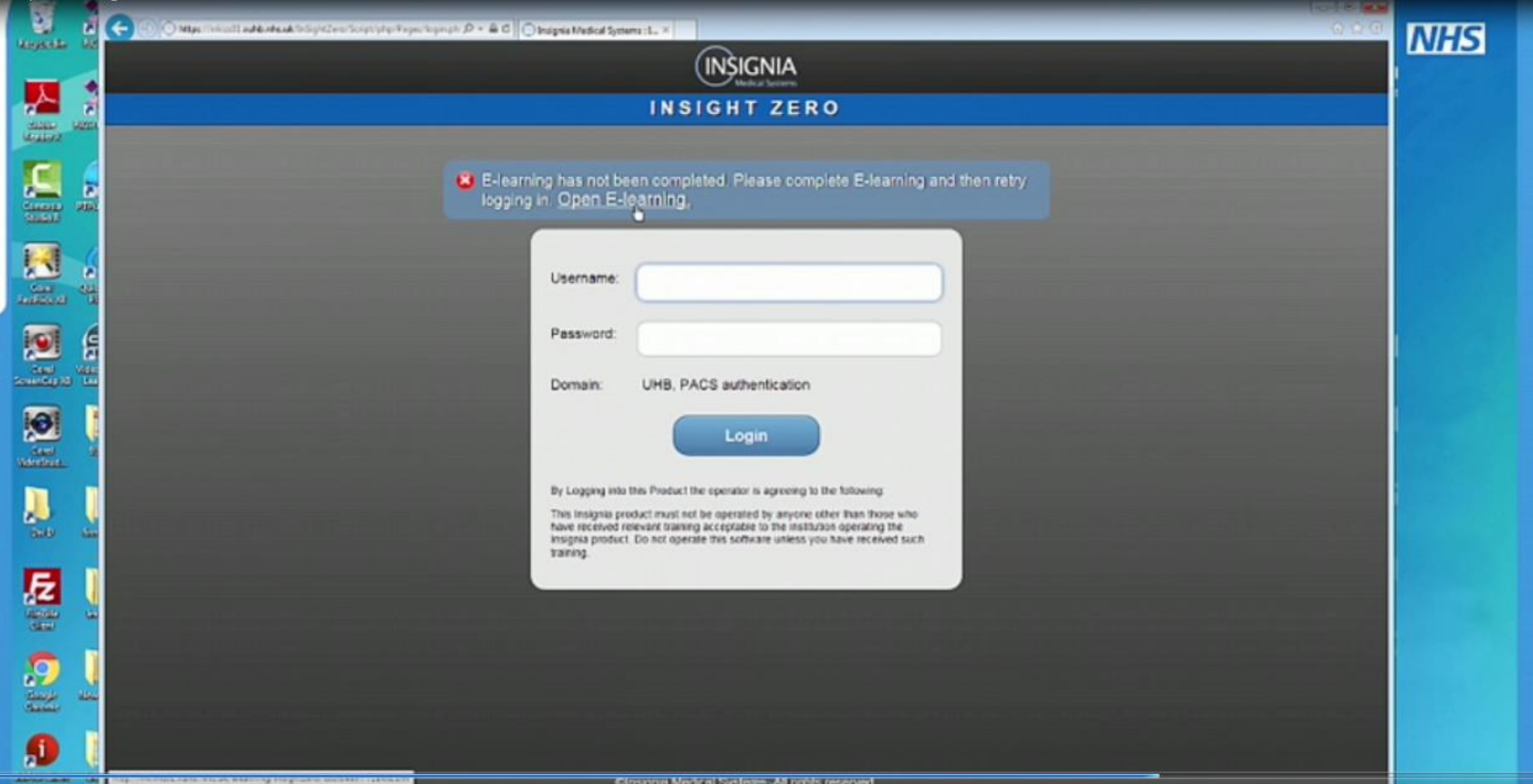

You would be able to use your Trust login details to use RISP.

The screenshot shows a web browser window displaying the InSight Zero E-Learning package. The browser's address bar shows the URL: <http://www.insigna.co.uk/learning/insightzero/2.0.3/index.html>. The page header includes the InSight Zero logo and the text "Training Videos". The main content area features a large "INSIGHT ZERO" logo and a prominent blue button that says "Click here to begin.". Below this, there is a copyright notice: "Copyright © 2017 ... Insignia Medical Systems Ltd. Insignia Medical System Ltd. All rights reserved. All intellectual property rights in the Software and documentation are owned by Insignia Medical Systems and are protected by United Kingdom copyright laws, Insignia Medical Systems Ltd or its licensors retain all rights not expressly granted." Contact information for Insignia Medical Systems Limited is provided, including their address in Basingsloke, United Kingdom, telephone and fax numbers, email address (info@insigniamedical.co.uk), and website (<http://www.insigniamedical.co.uk>). A CE mark with the number 0120 is displayed, along with a statement: "This symbol shows that the Insignia product is marketed as a medical device in the European Union and that it complies with all of the applicable requirements of the Medical Device Directive (93/42/EEC). The '0120' is the reference of the assessment body (Notified Body) SGS United Kingdom Ltd - 202B Worle Parkway, Weston-super-mare, BS22 6WA UK." The version number "E-learning Version 1.2.1 for InSight Zero 2.0.3.2" is also shown. On the left side of the browser window, a sidebar titled "InSight Zero E-Learning package." contains a "Contents" section with a list of training videos: "Training Videos", "How to use this guide", "Mouse and Finger Modes", "Header bar functions", "General Use of InSight Zero", "Logging On to InSight Zero", "Patient Search and report view", "Image placement", "Image manipulation", "Viewing in 3D", "Study Referral", and "Validate e-learning". The desktop background on the left shows various icons for applications like Microsoft Word, PowerPoint, and Google Chrome. On the right edge of the browser window, there is a vertical banner with the text "NHS Birmingham Foundation Trust" and the date "7th September".

The first time you login, you will be asked to complete a short E-Learning module.

This includes a set of videos that show you how to use the system and takes about 10 minutes to complete.

INSIGNIA
Medical Systems

INSIGHT ZERO

Username:

Password:

Domain: UHB, PACS authentication

Login

By Logging into this Product the operator is agreeing to the following:
This Insignia product must not be operated by anyone other than those who have received relevant training acceptable to the institution operating the Insignia product. Do not operate this software unless you have received such training.

NHS
Birmingham
Foundation Trust

7th September

ative
nest
able
tive
ctful

Once your E-learning is validated, you can start using the system.